




EZ Way, Inc.

EZ Way Sling with Head Support Sizing Chart



Sling Color Coding System

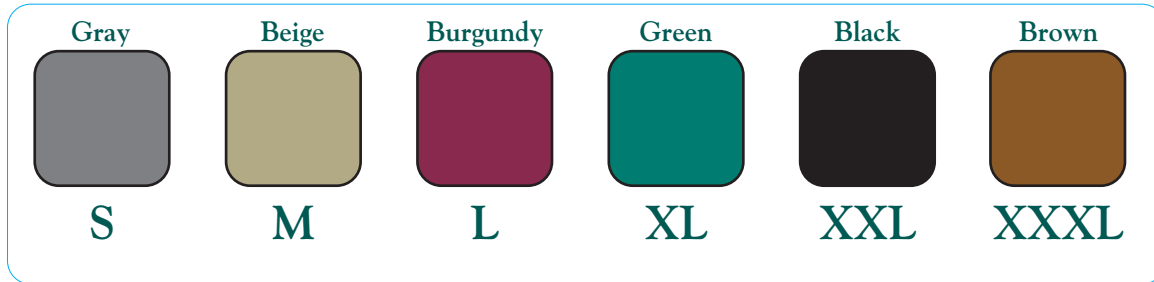

(Applies to washable and disposable slings only)

Sling Size	Small	Medium	Large	XL	XXL	XXXL
Weight of Patient	70-100 lbs.	90-220 lbs.	190-320 lbs.	280-450 lbs.	400-600 lbs.	600 + lbs.
Maximum distance from patient's tailbone to base of neck ³ <i>**Does not apply to Belted Mesh or Multi-Purpose slings**</i>	21"	24"	26"	29"	36"	37"

¹**NOTE!** The size/weight designations are merely estimates and basic guidelines. A proper fit will depend on factors other than weight measurements, including the height and girth of a patient. Fabric from one leg seam to the other should measure approximately the same as the width from the resident's hip to the other hip. A proper fit will involve the judgment of the caregiver.

^{*}It is important to evaluate the width of a patient in relation to the width of the sling, particularly the thighs and hips.
^{**}It is important that no portion of the patient overlap the sides of the sling.

²Color Coding is used on the binding of slings. It is not used on specialty slings.

³It is important that the base of the sling be positioned two inches below the coccyx (tailbone) and the top of the sling supports the top of the head. The crossed webbing should be at the nape of the patient's neck.

