

EZ Way Roll Board Competency Checklist

Purpose: To assist in the proper training of operating the EZ Way Roll Board.

Staff Name: _____ Date: _____ Observed by: _____

Yes No

- | | | |
|--------------------------|--------------------------|--|
| <input type="checkbox"/> | <input type="checkbox"/> | 1. Demonstrate how to properly position the two transferring surfaces prior to placing the Roll Board under the patient. |
| <input type="checkbox"/> | <input type="checkbox"/> | 2. Demonstrate how to properly position the Roll Board under the patient. |
| <input type="checkbox"/> | <input type="checkbox"/> | 3. Verify how many people should be involved in transferring a patient with the Roll Board. |
| <input type="checkbox"/> | <input type="checkbox"/> | 4. Demonstrate how to properly transfer patient from surface to surface using the Roll Board. |
| <input type="checkbox"/> | <input type="checkbox"/> | 5. Demonstrate how to remove the Roll Board out from under the patient. |
| <input type="checkbox"/> | <input type="checkbox"/> | 6. State how to properly clean and disinfect the product and how to remove turning sheet. |

For any questions or concerns, please refer to the operator's instructions.

ANSWERS: 1) Lock the wheels of any surface involved in a transfer, i.e. bed or stretcher and raise the bed rails on the exterior sides of each surface. The two interior surfaces should be the same height, should not be more than two inches apart, and ideally should be touching. 2) Log roll the patient away from the surface they are being transferred to and slide the board under the patient and draw sheet so at least 1/3 of the board is under the patient. 3) 2 caregivers. 4) The caregiver at the side of the surface the patient is being transferred to should grasp the draw sheet and pull while the caregiver on the opposite side nearest the patient pushes the patient. 5) A caregiver should log roll the patient and remove the EZ Way Roll Board. 6) A standard germicidal cleaner should be used to wipe down the inner Roll Board core and end caps. The sheet can be laundered and tumble dried. Do not use bleach on the product.