

WINNIPEG

Freestanding Bathing System





The RK12 Winnipeg is a freestanding tub that allows the caregiver 360° unobstructed access when bathing a resident.

The low threshold and wide side-entry door allows easy access for residents with various degrees of mobility and the built-in contoured seat allows the resident to sit comfortably and safely in a slightly reclined position. Designed to create the atmosphere of a luxury spa, the sleek exterior design of the RK12 Winnipeg will give your bathing area an upscale look.

Caregivers will enjoy the RK12 Winnipeg for its ease of use and the improved quality of care they can provide the resident. Featuring a full range of standard and optional features, the RK12 Winnipeg is designed to enhance the bathing experience for the resident.

The Rane Bathing & Accessibility product line provides a unique combination of style, ease of use, comfort, and low cost of ownership.



The built-in disinfection system

cabinet is conveniently located on the front side of the tub for easy access. The lockable cabinet provides a secure place for cleaning supplies and disinfectants.

The air jets on the Air Spa System

feature spring loaded check valves for back flow prevention, which eliminates the extra step to disinfect the air lines.



Standard Features

- Thermostatic Mixing Valve with anti-scald protection that dispenses 15 gallons per minute for fast tub fill.
- Built-in Disinfection System for disinfection of the bath's external surfaces minimizes the risk of cross-infection.
- Handheld Shower Wand for hair and upper body washing, which can be used at the same time the tub is filling.
- Analog Thermometer for monitoring tub-fill and showerwand temperatures.
- Lap and Shoulder Harness for resident safety.
- Secondary Door Lock to prevent accidental door opening when in use.
- 3/4" Supply Lines constructed of stainless steel.
- 24-hour Service Department.

Optional Features

 CleanRane[™] Air Spa System with variable speeds that enhance the therapeutic and relaxing effects of bathing for the user.

Why CleanRane™ Air Spa System?

The CleanRane $^{\text{TM}}$ Air Spa System enhances the therapeutic and relaxing effects of bathing and provides a pleasant experience for the resident. Hydromassage provides millions of preheated air bubbles in the water which increases stimulation of peripheral blood circulation, reduces pain and inflammation in the joints, gently massages away dirt, and induces deeper relaxation. The hygienic, low maintenance CleanRane™ Air Spa System features 12 air jets with spring loaded check valves for back flow prevention. Unlike most other whirlpool and air spa systems, the CleanRane™ Air Spa System does not allow water to enter the air lines, which can become a breeding ground for bacteria and mold. The motor is quiet, efficient, and has a ceramic heating element that blows warm air to help maintain the water temperature and warm the surface of the tub.

Technical Specifications

Length	61.5 inch
Width	33.5 inch
Height (to top of Door)	35 inch (lowest setting)
Height (to top of Console)	42.5 inch (lowest setting)
Adjustable Leveling Legs	2 inch max.
Seat Width (interior)	21.5 inch
Seat Height	19 inch (lowest setting)
Threshold Height	11 inch (lowest setting)
Transfer Device Access	7" H x 44" W
Weight Capacity (occupant)	400 lbs
Water Capacity (occupied)	48 gallons
Fill Time	3 minutes, 12 seconds
Drain Time	1 minutes, 58 seconds
Water Supply Inlet	3/4 inch diameter
Drain Size	1.5 inch
Electrical Supply	115V AC; 60Hz; 15 amp
Door Swing	Outward
Door Configuration	Right Hinge Only





The wide side-entry door and unobstructed mobile lift access will allow the resident to enter and exit the bathtub safely and easily.

Easy to Use Control Panel

Warranty: 5 years for external finishes and composite shell, 3 years for all moving and stationary parts, and lifetime on the door seal.

Certifications / Standards: A112.19.7-2012 / CSA B45.10-12 / UL 1795 / CSA C22.2 no. 218.2 / IAPMO Z124-2011 / CSA B45.5-11











EZ Way is proud to be a US manufacturer of Safe Patient Handling products



Form #4-011 Rev. 07/30/18