



Single Patient Use Glide™ Lateral Transfer System

Skills Checklist

Date: _____

Name: _____

Department: _____

Please mark each category with an "X" when completed

Task/Action	Complete	Initials
Identify the safety icons		
Describe the safety check associated with each icon		
Grab an assistant to help with the transfer		
Ensure both surfaces are flat and level		
Lay mat on one surface		
Make sure brakes are set on both platforms		
Ensure the patient is centered on the mat		
Buckle centering straps loosely		
Attach the hose to the mat and secure with the Hook and Loop strap		
Plug in blower		
Turn on the blower		
Grab the BackSmart® extended pull handles		
One caregiver guide the patient, the other caregiver step back and transfer patient		
Turn off and unplug the blower		
Detach the hose from the mat		

Comments:

