



EZ Way, Inc.  
710 E. Main St.  
P.O. Box 89 • Clarinda, IA 51632  
800-627-8940 • Fax 712-542-1899  
www.ezlifts.com

## SLINGS & HARNESSSES

### IMPORTANT NOTICE!!!!!!

This product is designed and manufactured to the highest possible performance specifications. It is constructed of high quality, durable, 100% synthetic fabrics. It has been individually inspected before shipping to ensure the safety of the product. However, water washing temperature, detergents and disinfectants, patient incontinence, frequency of use, types and weights of patients, etc., all have an impact on the life expectancy of each product. Because of these factors, the continued integrity of the product is not guaranteed. The institution or private user must therefore examine the product to ensure its integrity before each use.

EZ Way offers a 6-month warranty on slings and harnesses and recommends replacement after one year or if the sling or harness shows any sign of damage or wear. All slings and harnesses are capable of bearing a 1,000 lb. weight load, but must only be used to hold the amount of weight dictated by the lift or stand capacity.

Accordingly, the PURCHASER hereby accepts full responsibility for checking the condition of all slings and harnesses before each and every use on a patient or client.

---

***All EZ Way slings, harnesses and accessories are manufactured and tested for use with EZ Way equipment. If an item has been purchased to be used with another brand of product, EZ Way will not be held liable for any issues resulting from usage with non-EZ Way equipment.***

---

### EZ Sling and Harness Laundering Instructions

To get the longest life out of your product:

1. Do not bleach.
2. To prevent stains from setting, rinse 5 min. in 80° – 100° F. Stains will set when temp. is over 105° F.
3. Washing temp. 160° F. max.
4. RINSE THOROUGHLY in 100° F. If high alkaline detergent (with pH greater than 11.0) is used, rinse twice.
5. Tumble dry, temp. 100°F. max. High heat will weaken the fabric.
6. If applicable, snap the buckle together before washing and drying. This will prevent any damage to the plastic buckle.
7. If available, use a laundry bag to wash and dry the sling or harness.

### Wipeable EZ Sling and Harness Cleaning Instructions

EZ Way wipeable slings and harnesses contain an acrylic coating. They may be wiped down with any properly diluted EPA approved germicidal cleaning solution or quaternaries; alcohol (70% isopropyl of Ethyl). The cleaning solution should be sprayed on or applied with paper towels or disposable cleaning cloths using standard cleaning protocol. After removing all potentially infectious materials, wipe up excess solution and allow surface to dry. If sling or harness becomes badly soiled it can be laundered according to standard laundering instructions stated above. Temperature of water is not to exceed 160°F. It is recommended that the product be air-dried, but may be tumble dried on lowest setting. Do not iron.

**To order additional slings and harnesses - Please call 1-800-627-8940**

The State of California requires the following statement. WARNING: This product contains a chemical known to the State of California to cause cancer, birth defects or other reproductive harm.