



EZ Way, Inc.  
701 E. Washington St.  
P.O. Box 89 • Clarinda, IA 51632  
800-627-8940 • Fax 712-542-1899  
www.ezlifts.com

## Chest Strap User Instructions

The Chest Strap is an accessory designed for use with the EZ Stand. The Chest Strap is designed for attachment on the black padded handles of the EZ Stand arm (see photo inset). The medium Chest Strap model #50930 will only fit the 400 lb or 500 lb. capacity EZ Stand. The large Chest Strap model #50038 is designed specifically for use with the 800 lb. capacity EZ Stand.



### Attaching the Chest Strap to the EZ Stand

1. Raise the EZ Stand arm to a full, upright position.
2. With the strap-side of the chest strap facing away from you, grasp the shortest strap on either side of the pad. Approach the handle bars of the EZ Stand to attach.
3. With each buckle/receptacle in hand, reach around each handle bar and buckle the short straps behind the chest strap. Tighten the strap accordingly.
4. **USING WITH A PATIENT:** Once a resident/patient is in a standing position using the EZ Way harness, the caregiver may position and buckle the second-set of straps behind the patient/resident, over the harness. Use of these straps around a patient is an optional procedure.