




EZ Way, Inc.

# EZ Way Sling Sizing Chart



## Sling Color Coding System

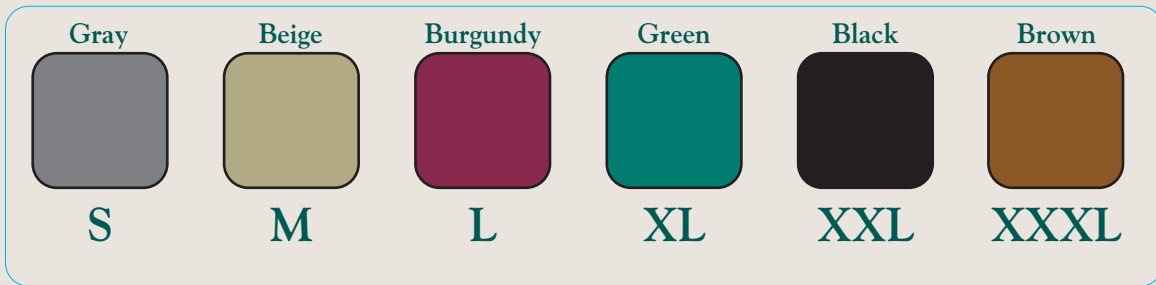

(Applies to washable and disposable slings.)

Sling Size	Small	Medium	Large	XL	XXL	XXXL
Weight of Patient	70-100 lbs.	90-220 lbs.	190-320 lbs.	280-450 lbs.	400-600 lbs.	600 + lbs.
Maximum distance from patient's tailbone to base of neck <sup>3</sup> <small>**Does not apply to Belted Mesh or Multi-Purpose slings**</small>	21"	24"	26"	29"	36"	37"


<sup>1</sup>**NOTE!** The size/weight designations are merely estimates and basic guidelines. A proper fit will depend on factors other than weight measurements, including the height and girth of a patient. A proper fit will involve the judgment of the caregiver.

\*It is important to evaluate the width of a patient in relation to the width of the sling.

\*\*It is important that no portion of the patient overlap the sides of the sling.

<sup>2</sup>Color Coding is used on the binding of slings. It is not used with specialty slings.

<sup>3</sup>It is important that the base of the sling be positioned two inches below the tailbone and the top of the sling is parallel with the top of the shoulder line (base of neck).

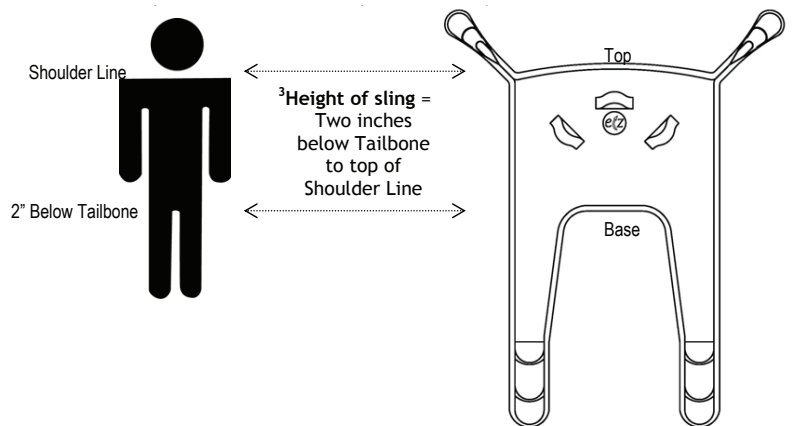

Form #2-150  
Rev. 9-25-14