

EZ Way Hydraulic Height Adjustable Reclining Shower Chair Competency Checklist

Purpose: To assist in the proper training of operating the EZ Hydraulic Height Adjustable Way Reclining Shower Chair.

Staff Name: _____ Date: _____ Observed by: _____

Yes No

- | | | |
|--------------------------|--------------------------|---|
| <input type="checkbox"/> | <input type="checkbox"/> | a.) Demonstrate how to lock and unlock the rear brakes. |
| <input type="checkbox"/> | <input type="checkbox"/> | b.) Demonstrate how to adjust the headrest. |
| <input type="checkbox"/> | <input type="checkbox"/> | c.) Demonstrate how to adjust the footplate height. |
| <input type="checkbox"/> | <input type="checkbox"/> | d.) Demonstrate how to raise and lower the armrests. |
| <input type="checkbox"/> | <input type="checkbox"/> | e.) Demonstrate how to prepare the chair to receive the patient / resident including locking the armrests, folding away the footrests, securing the chair from slipping away. |
| <input type="checkbox"/> | <input type="checkbox"/> | f.) Demonstrate how to raise and lower the chair using foot pump. |
| <input type="checkbox"/> | <input type="checkbox"/> | g.) Demonstrate how to adjust the chair into a reclined position and to bring the chair back to an upright position. |
| <input type="checkbox"/> | <input type="checkbox"/> | h.) Demonstrate how to remove the seat. |
| <input type="checkbox"/> | <input type="checkbox"/> | i.) Demonstrate the steps needed to use the commode pan. |
| <input type="checkbox"/> | <input type="checkbox"/> | j.) State the weight capacity of the unit. |