

Inserting Walker Handles

1. Insert the Walker Handles in the each of the handles located on the stand arm of the Smart Stand. (Figure 1)
2. Adjust the handles to the proper length needed for the individual. The handles are colored-coded to ensure they are adjusted equally. (Figure 2) Make sure each handle is adjusted to the same color and then tighten the walker handles by turning the knobs of the walker handles. (Figure 3)



Removing Walker Handles

1. Turn knob on walker handle in counterclockwise direction to loosen the walker handle from the stand arm handle.
2. Pull the walker handle out of the stand arm handle.



Please contact EZ Way at 1-800-627-8940 with any questions.