



EZ Way, Inc.

EZ Way Oxygen Tank Holder and IV Pole

The Oxygen Tank Holder and IV Pole is designed for use with Smart Stands.

Note: The Oxygen Tank Holder and IV Pole can be mounted to the following models: S400PN, S500PN, S400PS1, and S500PS1

- Allows for safe and efficient transfers with oxygen tank
- Provides support for IV bags when transferring or ambulating
- Designed for Smart Stands
- Easy to assemble and mount
- Fits C & D sized oxygen tanks



Item Number 11150

Oxygen Tank Holder
And IV Pole Kit

EZ Way, Inc. • PO Box 89, 807 E. Main • Clarinda, IA 51632 • 1-800-627-8940

⚠WARNING: Cancer and Reproductive Harm - www.P65Warnings.ca.gov.

Form #2-359
02/23/21



EZ Way Oxygen Tank Holder and IV Pole Installation Instructions

EZ Way, Inc.

The Oxygen Tank Holder and IV Pole is designed for use with Smart Stands.

Note: The oxygen tank holder can be mounted to the following models: S400PN, S500PN, S400PS1, and S500PS1.

Parts for Item Number 11150

Oxygen Tank Holder And IV Pole Kit




Oxygen Tank/IV Pole Bracket



5/16" x 3/4" bolts and 1/4" x 20 lock nuts



Height Adjustable IV Pole

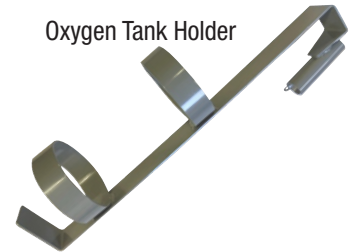

Oxygen Tank Holder

Installation Instructions:

1. Mount the Oxygen Tank/IV Pole Bracket even with the bottom of the Battery Receiver Housing.
2. Insert the 5/16" x 3/4" bolt into top holes of the Oxygen Tank/IV Pole Bracket and secure it by using 1/4" x 20 nylon lock nut. Repeat this step for the bottom holes. (Tools required: 7/16" box end wrench and 5/16" Allen wrench)
3. Align the slot in the Oxygen Tank Holder so it will accept the keyway on the Oxygen Tank/IV Pole Bracket. Lower the Oxygen Tank Holder onto the Pole Bracket.
4. Assemble the IV Bag Holder by threading the bolt into the adjustable height IV Pole.
5. Slide the IV Pole into the Oxygen Tank/IV Pole Bracket as shown.

