

# EZ Way Reclining Shower Chair Competency Checklist

Purpose: To assist in the proper training of operating the EZ Way Reclining Shower Chair.

Staff Name: \_\_\_\_\_ Date: \_\_\_\_\_ Observed by: \_\_\_\_\_

Yes      No

- |                          |                          |  |
|--------------------------|--------------------------|--|
| <input type="checkbox"/> | <input type="checkbox"/> | a) Demonstrate how to lock and unlock the rear brakes.   |
| <input type="checkbox"/> | <input type="checkbox"/> | b) Demonstrate how to adjust the headrest.   |
| <input type="checkbox"/> | <input type="checkbox"/> | c) Demonstrate how to adjust the footplate height.   |
| <input type="checkbox"/> | <input type="checkbox"/> | d) Demonstrate how to raise and lower the armrests.  |
| <input type="checkbox"/> | <input type="checkbox"/> | e) Demonstrate how to prepare the chair to receive the patient / resident including locking the armrests, folding away the footrests, securing the chair from slipping away. |
| <input type="checkbox"/> | <input type="checkbox"/> | f) Demonstrate how to adjust the chair into a reclined position and to bring the chair back to an upright position.  |
| <input type="checkbox"/> | <input type="checkbox"/> | g) Demonstrate how to remove the seat.   |
| <input type="checkbox"/> | <input type="checkbox"/> | h) Demonstrate the steps needed to use the commode pan.  |
| <input type="checkbox"/> | <input type="checkbox"/> | i) State the weight capacity of the unit.  |