



EZ Way Stand Aid | Usage Guidelines

EZ Way, Inc.

Please note: It is always the responsibility of the facility to assess the acuity of the patient, per facility protocol, and to determine if they fit the criteria.

Description and Uses

The EZ Way Stand Aid is a manual standing aid to allow patients to assist themselves in preparation for transferring. The EZ Way Stand Aid is positioned between a traditional walker and electric stand assist lift in terms of function.

The EZ Way Stand Aid is also a transfer assist unit which keeps residents active and engaged in the transferring process. Users simply grasp the middle bar and pull themselves up. A padded split seat swings out for loading or unloading then swings back and locks to form a comfortable, secure seat for transport.

Appropriate Usage Criteria

Patients qualified to use the Stand Aid should have:

- 1) Adequate arm strength to pull themselves upward
- 2) Enough leg strength to support their own weight

Patients who meet these criteria listed in numbers 1 and 2 above and have difficulty walking will find the Stand Aid a useful and safe transferring device. For residents lacking the strength to assist themselves, an EZ Way Smart Stand or EZ Way Smart Lift is recommended. This determination should be considered based on the patient's condition, and the assessment of clinical staff.

Types of Transfers Used with the EZ Way Stand Aid

The EZ Way Stand Aid is capable of the following transfers when used according to the instructions:

- 1) Bed to Chair/Wheelchair
- 2) Bed/Chair/Wheelchair to commode
- 3) Room to Room
- 4) From vehicle or into vehicle



- STA450** - EZ Way Stand Aid with Movable Legs
- 500928** - Sliding Belt Medium
- 500935** - Sliding Belt Large
- 1915** - Safety Belt



Instructional Video

Scan the QR code to watch the EZ Way Stand Aid instructional video or go to [youtube.com/ezwayinc](https://www.youtube.com/ezwayinc).