



EZ Way, Inc.

# BRIDGEROLLER™

Safely transfer patients from one surface to another in seconds

## PREFACE

Before using this product, we suggest you read all the information in this User Guide. The products design and improvements are compliant with the requirements of the Medical Device Directory 93/42/EEC. These instructions are a guide only and do not replace protocols and procedures as devised by your manual handling coordinator.

## PRODUCT DESCRIPTION

**BridgeRoller™** is a lightweight, foldable, patient transfer device which transfers patients between a bed and x-ray table, operating table, shower trolley or ambulance stretcher. It is effectively a patient transfer board and padded slide tube in one.

## SAFETY INFORMATION

- Before transferring patient, be sure both bed and stretcher, trolley, tables are securely locked
- Be sure both bed and stretcher, trolley tables are leveled when placing *BridgeRoller™* on them. The maximum safety gap is 15cm (6") between two surfaces for transfer however it is strongly recommended that all surfaces are leveled when possible
- Do not sterilize *BridgeRoller™* in high temperature
- When rolling *BridgeRoller™*, be sure patients hair is positioned on the outer layer and away from the rolling to edge to prevent hair rolling up during operation
- Clean the *BridgeRoller™* using a surface disinfectant
- SWL 660lbs when used with 6" gap

## BRIDGEROLLER™ USER INSTRUCTIONS - WE RECOMMEND 2 USERS MINIMUM



1 Lean patient to one side and roll *BridgeRoller™* under patient



2 Gently pull draw sheet and *BridgeRoller™* simply rolls across gap from bed to bed



3 Patient has now been transferred with no lifting



4 Lean patient to one side and roll *BridgeRoller™* away from patient for easy removal.


**Please note:** These instructions are a guide only and do not replace protocols and procedures as devised by your manual handling coordinator. Always consult with your manual handling coordinator or seek professional advice before use. Never leave patients unattended on the *BridgeRoller™*. For demonstration purposes, the guideline above shows the *BridgeRoller™* used with a transfer board. The *BridgeRoller™* can be used without a transfer board if the gap between the 2 transfer surfaces is less than 6".

For more information please email: [sales@ezlifts.com](mailto:sales@ezlifts.com)

EZ Way, Inc. • PO Box 89, 807 E. Main • Clarinda, IA 51632 • 1-800-627-8940

Form #2-354  
Rev. 08/24/21

**⚠WARNING:** Cancer and Reproductive Harm - [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).

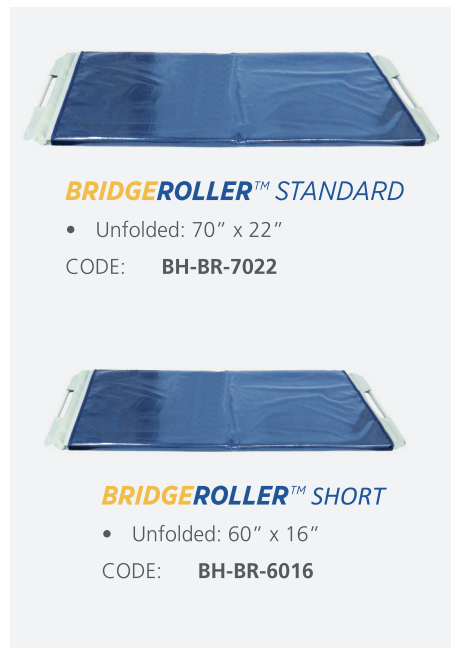

### BRIDGEROLLER™ STANDARD

- Unfolded: 70" x 22"
- CODE: **BH-BR-7022**

### BRIDGEROLLER™ SHORT

- Unfolded: 60" x 16"
- CODE: **BH-BR-6016**