



EZ Way, Inc.

"Your Total Safe Patient Handling Solution"

500898 Padded Wrist Strap Instructions

The padded wrist strap can be used on EZ Smart Stands to hold the arm of a stroke patient in position during a transfer.



1. The strap should be attached to the lower handle of the Smart Stand before attaching the harness to the machine.



2. Slide the strap over the lower handle of the Smart Stand arm with the Item Number Tag facing to the outside of the arm. The wrist strap should be slid all the way to the bend in the lower handle.
3. Following the Smart Stand operating instructions, attach the EZ Way harness to patient and then attach harness loops to the Smart Stand.
4. While the Smart Stand arm is in the down position, assist patient and place hand on lower handle of the Smart Stand arm. Secure wrist by wrapping free end of wrist strap around the patient's wrist and fasten loose end to the white Velcro patch.

5. Secure wrist by placing the patient's thumb into the padded side of the wrist strap thumb hole (palm down) at the top of the wrist strap. Gently roll the patient's hand up and out. Secure the velcro to the wrist strap.



6. After the transfer is complete, remove the wrist strap from the patient's wrist.